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BATTERY FOR DEA CARDIAC POWER HEART G3 EQUIPMENT

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DESCRIPTION

Medical grade IntelliSense lithium battery to power your Powerheart AED. Easy to maintain, it ensures your device is ready to provide professional defibrillation. For use with the G3 AED.

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CHARACTERISTICS

Output voltage: 12 V DC. Non-rechargeable batteries. Lithium content: 9.2 g. Number of downloads: 290 downloads.

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WARRANTY

If there is any manufacturing defect or inconformity you can contact the nearest distributor or write us directly to: contacto@steelprosafety.com. The distributor is not liable to any injury, grievance or personal or patrimonial detriment resulting from improper use of the product. Before using this device make sure it is suitable, adequate or safe for the use intended.

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WARNINGS

Failure to follow the manufacturer's recommendations may cause equipment damage. Highly aggressive environments can cause component wear, which is why it is necessary to constantly check the condition of the equipment. In case of recharging the battery, there is a risk of explosion or fire. Pressurized content. Do not recharge, short circuit, puncture, deform, or expose to temperatures above 65°C (149°F). Remove the battery when it is discharged. Please recycle or dispose of the lithium battery according to federal, state and local laws. To avoid the risk of fire and explosion, do not burn or incinerate the battery.

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USES

For AED Cardiac Power Heart G3 equipment.